


Pressure vs. Flow Rates for ResMed Masks

This table and corresponding graph show the airflow rates through mask vents at varying mask pressures.



Pressure (cm H ₂ O)	Nasal Masks				Nasal Pillows			Full Face Masks	
	Mirage™ FX Mirage FX for Her Mirage SoftGel™ Mirage Activa™ LT Mirage Micro™ Mirage Activa	Mirage Kidsta™	Pixi™ ¹	Ultra Mirage™ II	Swift™ LT Swift LT for Her	Swift FX Swift FX for Her Mirage Swift™ II	Quattro™ FX ² Quattro FX for Her ² Mirage Quattro™ Ultra Mirage™ Mirage Liberty™		
4	19	21	20	18	21	20	22		
6	24	26	25	24	25	25	28		
8	28	30	29	29	28	29	32		
10	31	33	32	34	31	33	37		
12	34	36	38	38	33	37	41		
14	37	39	40	43	36	40	44		
16	40	42	42	46	38	43	48		
18	43	45	46	50	40	46	51		
20	45	47	49	53	42	49	54		

Expected flow rates (Liters/min)

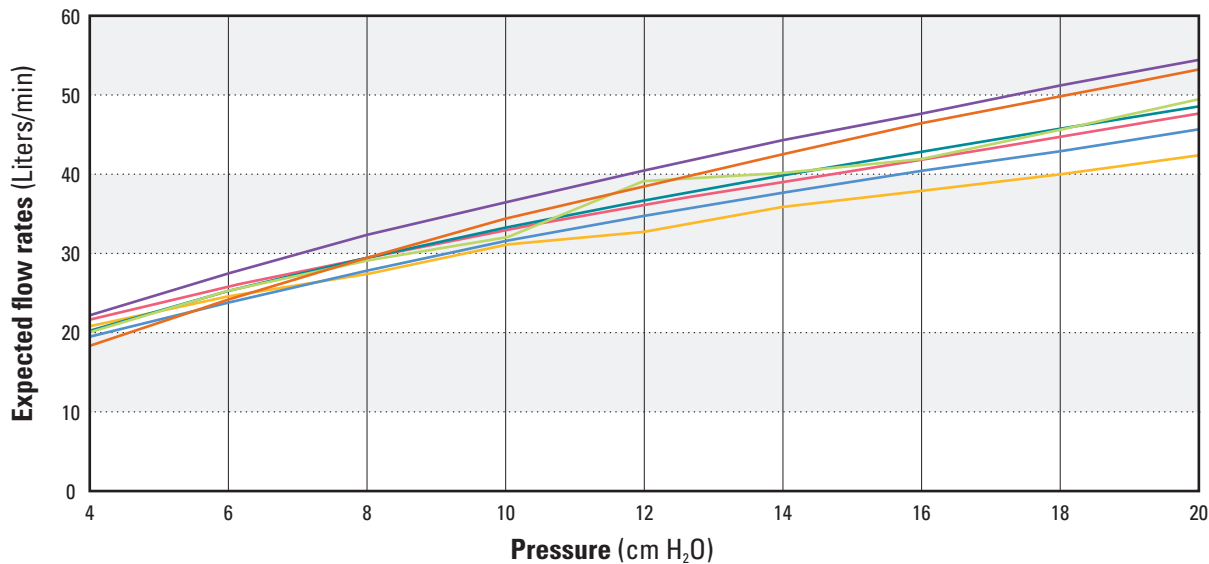

- Quattro FX, Quattro FX for Her, Mirage Quattro, Ultra Mirage, Mirage Liberty
- Ultra Mirage II
- Pixi
- Swift FX, Swift FX for Her, Mirage Swift II
- Mirage Kidsta
- Mirage FX, Mirage FX for Her, Mirage SoftGel, Mirage Activa LT, Mirage Micro, Mirage Activa
- Swift LT, Swift LT for Her

Note: The pressure flow data is the nominal specification curve, rounded to the nearest L/min. As a result of manufacturing variations, the vent flow rate may vary.



¹ Pixi therapy pressure range 3-20 cm H₂O. See User Guide for pressure flow rates.
² Quattro FX therapy range 4-30 cm H₂O. See User Guide for pressure flow rates.



Mask / Flow Generator Settings

The table below indicates which mask setting to use with your ResMed flow generator depending on which ResMed mask you are using.

Patient Interface	S9™ Flow Generator Settings	S8™, S8 II Flow Generator Settings	Stellar™ 100 Settings	VPAP™ Flow Generator Settings						
				VPAP III	VPAP III ST-A	Malibu	VPAP Adapt SV	VPAP Auto 25	VPAP S / VPAP ST	VPAP Tx
Swift LT, Swift LT for Her, Mirage Swift II, Swift FX, Swift FX for Her Nasal Pillows	Pillows	Swift	Pillows	Mirage	Mirage	Swift	N/A	Swift	Swift	Pillows
Mirage FX, Mirage FX for Her Nasal Masks	Nasal	Standard	Nasal	Standard	Standard	Vista	Vista	Standard	Standard	Nasal
Mirage Micro Nasal Mask	Nasal	Standard	Nasal	Standard	Standard	Meridian	Vista	Standard	Standard	Nasal
Mirage Activa, Mirage Activa LT, Mirage SoftGel Nasal Masks	Nasal	Activa	Nasal	Standard	Standard	N/A	Activa	Activa	Activa	Nasal
Ultra Mirage II Nasal Mask	Nasal Ultra	Ultra	Ultra	Ultra	Ultra	Ultra	Ultra	Ultra	Ultra	Nasal Ultra
Mirage Kidsta, Mirage Micro for Kids Nasal Masks	N/A	N/A	Pediatric	N/A	Standard	N/A	N/A	N/A	N/A	N/A
Quattro FX, Quattro FX for Her, Ultra Mirage, Mirage Quattro, Mirage Liberty Full Face Masks	Full Face	Mir Full	Full Face	Mir Full	Mir Full	Full Face	Full Face	Mir Full	Mir Full	Full Face
Pixi	N/A	N/A	Pediatric	N/A	N/A	N/A	N/A	N/A	N/A	N/A

For further information regarding ResMed products, please call Customer Service at 800.424.0737.