


## Activa™ LT

### NASAL MASK

#### FITS RIGHT FOR AN EASY NIGHT

The latest nasal mask from ResMed is the only mask with ActiveCell™ technology. The Activa™ LT's unique cushion features an "active cell" that expands and contracts to accommodate movement and therapy pressure changes throughout the night. The mask creates a gentle, secure seal and a dynamic defense against mask leak.

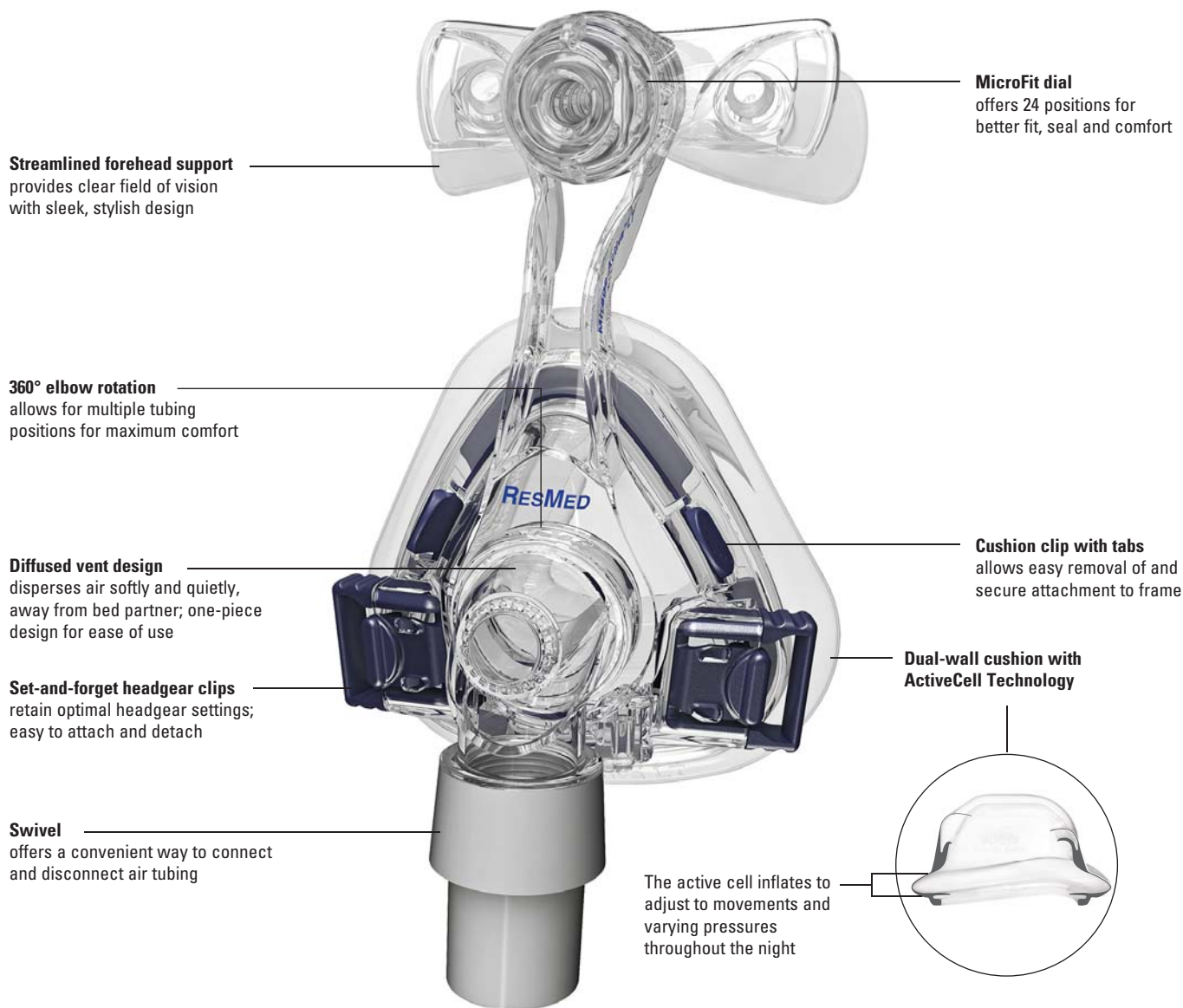

# TECHNOLOGY HIGHLIGHT

## Activa LT

Same superior Mirage Activa™ sealing technology ... now lighter and more streamlined

Dynamic cushion and MicroFit dial combination ... fitting is quicker and easier than ever

One-piece diffused vent design ... overall quiet performance




### A fine-tuned fit

Because the cushion's active cell does all the work, the mask is less dependent on headgear tension to achieve and maintain an effective seal. In fact, the Activa LT works best with some slack in the headgear—no more overtightening straps! That means quicker fitting time, less callbacks and improved user compliance.



With only minor fitting adjustments required, the MicroFit dial provides 24 positions of comfort at your fingertips. Easily adjustable with just one hand (even while lying down), the dial fine-tunes the fit where users need it most.

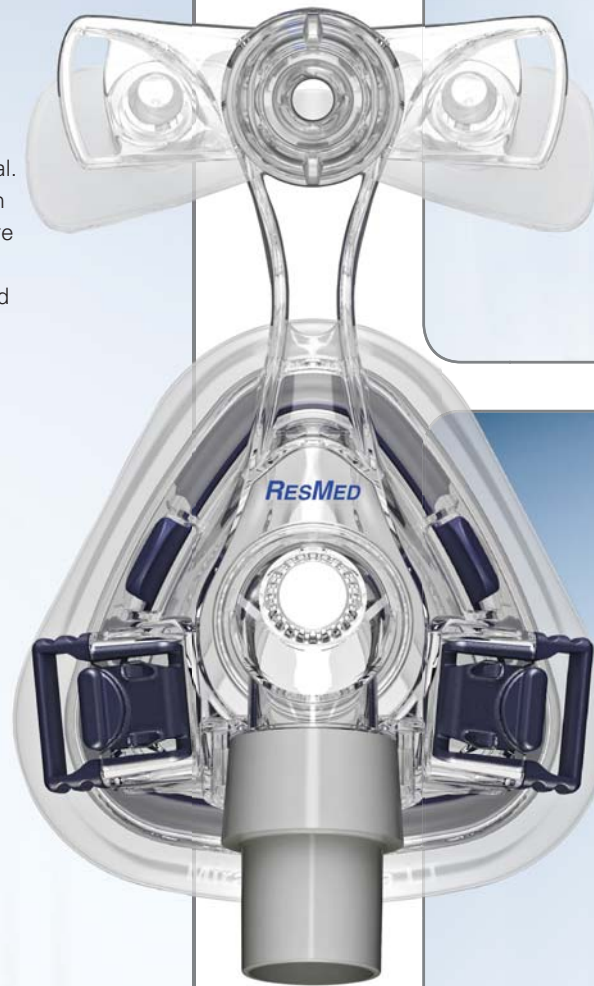

### ActiveCell for peace of mind

The Activa LT cushion is not your average cushion. Once therapy is switched on, the cushion is drawn to the face, creating a gentle yet secure seal. As users move or their therapy pressure changes throughout the night, the cushion's active cell automatically expands and contracts, delivering the most effective, comfortable seal.






ActiveCell expands ...

... and contracts

### Room to breathe, freedom to move

Because the Activa LT maintains its seal throughout the night, users are free to move around during sleep. The mask automatically, continually and actively adjusts to movement and other changes during sleep, providing a dynamic defense against leak.

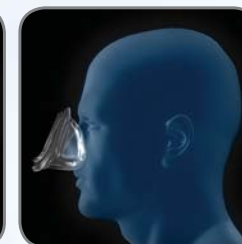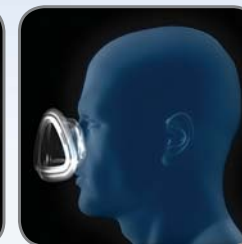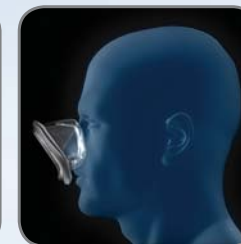

### Quiet comfort

Featuring an innovative vent design, the Activa LT is not only sophisticated, but also quiet. Located on the elbow for easy cleaning and disassembly, the diffused vents are designed to disperse air quietly and softly, at a short distance, away from the bed partner. The elbow partition also aids CO<sub>2</sub> washout and further reduces noise.

### PRODUCT CODES

#### Mirage Activa LT Nasal Mask

Americas	
Medium	60148
Large	60149
Large Wide	60150

### REIMBURSEMENT CODES

#### Mirage Activa LT Nasal Mask

HCPCS	Code Descriptor
A7034	Nasal mask used with positive airway pressure device, 1 per 3 months
A7032	Replacement cushion for nasal mask, 2 per month
A7035	Headgear used with positive airway pressure device, 1 per 6 month

